

KODAK EC-L FILM SYSTEM TECHNIQUE CHART

Assumes source to film distance of: 105-115 cm for AP or PA; 100 cm SAD
130-140 cm for Laterals; 100 cm SAD

Kodak EC Film
Item 113-234
Special High Contrast

**EC-L ONCOLOGY
CASSETTE**
Item 113-232
Regular Speed Cassette

**EC-L FAST
CASSETTE**
Item 113-233
30 % Faster Cassette

PATIENT PROFILE

MONITOR UNITS

MONITOR UNITS

LUNG / CHEST

| | | 4MV | 6MV | 10MV | 18MV | 4MV | 6MV | 10MV | 18MV |
|---------|------|-----|-----|------|------|-----|-----|------|------|
| AP/PA | Most | 1+3 | 1+2 | 1+2 | 1+2 | | | | 1+1 |
| OBLIQUE | Most | 1+5 | 1+4 | 1+3 | 1+4 | | | | 1+2 |

PELVIS

| | | 4MV | 6MV | 10MV | 18MV | 4MV | 6MV | 10MV | 18MV |
|---------|----------------------------------|-----|-----|------|------|-----|-----|------|------|
| AP / PA | Small / Medium | 1+4 | 1+3 | 1+2 | 1+2 | 1+2 | | | 1+1 |
| AP / PA | Large | 1+5 | 1+4 | 1+3 | 1+4 | 1+4 | | | 1+2 |
| LATERAL | Small | | | | | 2+5 | 2+5 | 1+5 | 1+4 |
| LATERAL | Medium | | | | | 2+7 | 2+6 | 1+6 | 1+5 |
| LATERAL | Large | | | | | 2+8 | 2+7 | 2+6 | 2+5 |
| LATERAL | Small / Medium (Single Exposure) | | | | | 8 | 7 | 6 | 6 |
| LATERAL | Large (Single Exposure) | | | | | 10 | 8 | 7 | 7 |

BREAST

| | | 4MV | 6MV | 10MV | 18MV | 4MV | 6MV | 10MV | 18MV |
|----------|-----------------|--------|--------|------|--------|-----|-----|------|--------|
| TANGENTS | Small | 1+2 | 1+2 | 1+2 | 1+2 | | | | 1+1 |
| TANGENTS | Medium / Large | 1+4 | 1+3 | 1+3 | 1+3 | | | | 1+2 |
| TANGENTS | Single Exposure | 3 or 4 | 3 or 4 | 3 | 3 or 4 | 4 | | | 2 or 3 |

HEAD / NECK / BRAIN

| | | 4MV | 6MV | 10MV | 18MV | 4MV | 6MV | 10MV | 18MV |
|---------|------|-----|-----|------|------|-----|-----|------|------|
| AP / PA | Most | 1+3 | 1+2 | 1+2 | 1+2 | | | | 1+1 |
| LATERAL | Most | 1+4 | 1+3 | 1+3 | 1+3 | | | | 1+2 |

SHOULDER / CLAVICLE

| | | 4MV | 6MV | 10MV | 18MV | 4MV | 6MV | 10MV | 18MV |
|------------|------|-----|-----|------|------|-----|-----|------|------|
| AP / PA | Most | 1+3 | 1+2 | 1+2 | 1+2 | | | | 1+1 |
| SUPER CLAV | Most | | 1+3 | | | | | | |

ABDOMEN

| | | 4MV | 6MV | 10MV | 18MV | 4MV | 6MV | 10MV | 18MV |
|---------|--------|-----|-----|------|------|-----|-----|------|------|
| LATERAL | Medium | 4+8 | | | | 3+7 | 2+6 | 1+6 | 1+5 |
| OBLIQUE | Medium | 2+7 | 2+5 | 1+5 | 2+5 | 2+5 | | | 1+4 |

KODAK is a Trademark of Eastman Kodak Company

INFO. 113-232-A 03-00

Radiation Products Design, Inc. □ 5218 Barthel Industrial Drive □ Albertville, MN 55301
(763) 497-2071 ♦ Fax: (763) 497-2295 ♦ www.rpdinc.com ♦ Email: sales@rpdinc.com