



Radiation Products Design, Inc.

Item 467-490 Baumrucker Urinary Incontinence Clamp

Washing Instructions:

- Wash with a mild soap in warm water
- Rinse thoroughly in cool clean water
- Gently squeeze foam to discharge excess water
- Let unit air dry in cool environment away from excess heat or direct sunlight
- Do not use bleach, detergent, hot water, washer/dryer or blowdryer

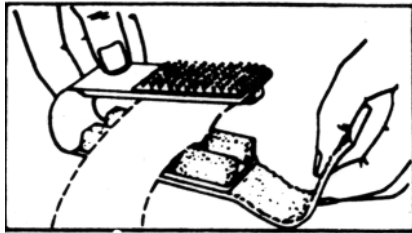


Fig. 1

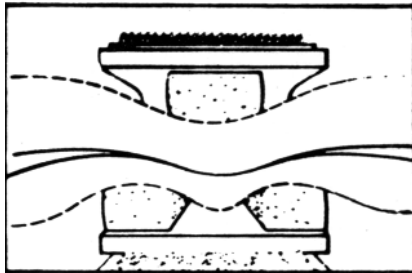


Fig. 2

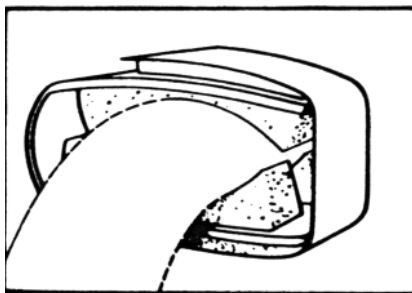


Fig. 3

Instructions For Use

1. Apply talcum powder to skin of penis, and to clamp pads before applying.

2. Place penis between the two open sides of the clamp. Be sure the two rubber pads are beneath the penis. See figure 1 and 2. Take care not to include pubic hair in the clamp.

3. Then pull Velcro strap over and press to top side of clamp. Make sure that resulting pressure on penis is just enough to stop dribbling - DO NOT OVER COMPRESS. See figure 1 and 3.

4. Release clamp at least every 2 - 3 hours to drain bladder, and reapply following above steps 2 thru 4.

5. When resting or sleeping, clamp should be applied with less pressure than when upright and active.

6. Keep clamp dry and clean. Alternate with a spare at least every 24 hours. Repeat step 1 above before re-using clamp.

7. This product should be used only on the advice, and under the supervision of a physician. This clamp is not intended for use in conjunction with other incontinence devices, or with indwelling catheters. This device should not be used in proximity with open sores.

5218 Barthel Industrial Drive N.E. Albertville, MN 55301
Phone: 763-497-2071 / 800-497-2071 Fax: 763-497-2295
www.rpdinc.com e-mail: sales@rpdinc.com