



Expect Service

Radiation Products Design Inc

INSTRUCTIONS

RPD INFORMATION

Address 5218 Barthel Industrial Drive
Albertville, MN 55301

Website www.rpdinc.com

Email sales@rpdinc.com

Phone 763-497-2071 or 800-497-2071

Fax 763-497-2295

RPD PRODUCT INFORMATION

Item Number	Description
707-000	Vaginal Depth Scale w/Introitus Marker 1.6 cm Diameter x 25 cm Long
707-120	Vaginal Depth Scale 2 cm Diameter x 26 cm Long
707-125	Vaginal Depth Scale 2.5 cm Diameter x 26 cm Long
707-132	Vaginal Depth Scale 3.2 cm Diameter x 26 cm Long
707-138	Vaginal Depth Scale 3.8 cm Diameter x 26 cm Long
707-145	CT Vaginal Depth Scale 1.6 cm Diameter x 26 cm Long

DISCLAIMER

THESE PRODUCTS ARE NOT STERILE AND ARE TO BE USED BY AUTHORIZED PERSONNEL ONLY.

RADIATION PRODUCTS DESIGN INC assumes no liability for consequential damages of any kind for this material when used interchangeably with products of other manufacturers/suppliers or for any direct or indirect results and consequences of its use or misuse by the customer.

CLEANING

Cleaning: Use detergent similar to Klenzyme®, Manu-Klenz® or NpH-Klenz® and water.

STERILIZATION

Sterrad 100NX Use Hydrogen Peroxide solution; temperature must not exceed 140° F.

Steris System 1 Use Parasitic Acid solution; temperature must not exceed 140° F.

WARNING DO NOT AUTOCLAVE

ACCESSORIES

Item #	Description
460-006	Latex Cover for Vaginal Markers 2 cm Diameter x 30 cm Long
707-010	Introitus Marker ONLY for Item 707-000
707-020	Vaginal Depth Scale Holder, Standard
707-030	Vaginal Depth Scale Holder, Large
707-120-6	Introitus Marker ONLY for Item 707-120
707-125-6	Introitus Marker ONLY for Item 707-125
707-132-6	Introitus Marker ONLY for Item 707-132
707-138-6	Introitus Marker ONLY for Item 707-138

**This Page Intentional
Left Blank**