



# Radiation Products Design, Inc.

## Testicle Shield and Adjustable Stand Items 928-200 through 928-505

### Instructions for Use

1. After patient is in position on the couch, place the adjustable stand between the patient's thighs. The stand should be placed so the shield holder ring is towards the patient and the counter-weighted base is towards the feet.
2. Adjust the ring height to scrotum.
3. If using a scrotum cover or a soft cotton sock have patient put it on.
4. Attach the two rubber straps to the knob on one half of the testicle shield, straps should be perpendicular to scrotal opening. The use of the rubber straps is optional.
5. Place the half of the testicle shield with the rubber straps on the ring holder with the knob going down towards couch, the shield opening facing up and cutout towards scrotum.
6. Move the adjustable stand towards the patient and have the patient place scrotum into the shield at the cutout area.
7. Adjust the height of the shield by turning the knob on the top of the stand to a level comfortable for patient.
8. Place top half of testicle shield over scrotum and onto bottom half of testicle shield, taking care not to pinch the scrotum.
9. Make adjustments to height of shield and scrotum opening for patient comfort.
10. Pull the rubber straps up and attach to the knob on top half of testicle shield.
11. Rotating the testicle shield so the scrotum entrance is toward the ceiling will reduce scatter radiation from entering the shield.

### Cleaning Instructions

The Adjustable stand can be cleaned with soap and water and/or low or intermediate level disinfectants.

The Testicle Shields can be cleaned with soap and water and intermediate or high level disinfectants.

**5218 Barthel Industrial Drive NE – Albertville, MN 55301**  
**Phone: 763-497-2071 or 800-497-2071 Fax: 763-497-2295**  
**www.rpdinc.com sales@rpdinc.com**