



**Radiation Products Design, Inc.**

**Item 970-650-21 through 970-650-32 ABS Ovoid Caps with Tungsten**

**This Product is not sterile**

**ABS Ovoid Cap Fit:** The ovoid caps should fit snug on the applicator. If the caps are too loose sterile KY lubricating jelly (or comparable lubricant) may be used inside the cap to provide a suction effect. ABS stands for Acrylonitrile butadiene styrene.

**Recommended Sterilization Procedure:** Ethylene Oxide (ETO) Temperature between 120 to 140 degrees Fahrenheit. Time: 2-3 hours. Aeration time: 8-12 hours.

**Do Not Autoclave**

**Sterrad has not been tested and therefore should not be used.**

**To be used by authorized personnel only.**

Radiation Products Design, Inc. assumes no liability or for consequential damages of any kind for this material or any direct or indirect result and consequences of its use or misuse by the customer.

**5218 Barthel Industrial Drive NE – Albertville, MN 55301  
Phone: 763-497-2071 or 800-497-2071 Fax: 763-497-2295**