



Expect Service

Radiation Products Design Inc

---

## INSTRUCTIONS

---

### RPD INFORMATION

<b>Address</b>	5218 Barthel Industrial Drive Albertville, MN 55301
<b>Website</b>	<a href="http://www.rpdinc.com">www.rpdinc.com</a>
<b>Email</b>	<a href="mailto:sales@rpdinc.com">sales@rpdinc.com</a>
<b>Phone</b>	763-497-2071 or 800-497-2071
<b>Fax</b>	763-497-2295

### RPD PRODUCT INFORMATION

<b>Item Number</b>	<b>Ovoid Caps, High Temperature Plastic</b>
970-650-40	Right, 2.0cm, with Tungsten
970-650-41	Left, 2.0cm, with Tungsten
970-650-42	Right, 2.5cm, with Tungsten
970-650-43	Left, 2.5cm, with Tungsten
970-650-44	Right, 3.0cm, with Tungsten
970-650-45	Left, 3.0cm, with Tungsten

**THESE PRODUCTS ARE NOT STERILE AND  
ARE TO BE USED BY AUTHORIZED  
PERSONNEL ONLY.**

RADIATION PRODUCTS DESIGN INC  
assumes no liability for consequential damages  
of any kind for this material when used or for  
any direct or indirect consequences of its use  
or misuse by their customer.

## VOID CAP FIT

The high temperature plastic ovoid cap should fit snug on the applicator.

## CLEANING

Use detergent similar to Klenzyme®, Manu-Klenz® or NpH-Klenz® and water. Ultrasound cleaner may be used.

## STERILIZATION

Steam Autoclave at normal cycle.

**This Page Intentional  
Left Blank**