



Expect Service

Radiation Products Design Inc

INSTRUCTIONS

RPD INFORMATION

Address	5218 Barthel Industrial Drive Albertville, MN 55301
Website	www.rpdinc.com
Email	sales@rpdinc.com
Phone	763-497-2071 or 800-497-2071
Fax	763-497-2295

RPD PRODUCT INFORMATION

Item Number	Description
182-201	Stirrups for 182-000


INSTRUCTIONS FOR MOUNTING

1. Unpack and check the contents. Ensure that the following parts are enclosed.

Quantity	Description
2	Stirrups
2	Bent Rods
2	Knobs

NOTE: The accessories can be mounted on most Biodex Deluxe Ultrasound Tables with black cushions or in any holder that will accommodate a 0.5" diameter (1.3 cm) pole.

2. Place the rods into the holes in the stirrup mounting brackets. Adjust the rod height with the tightening knobs (see Figure below).

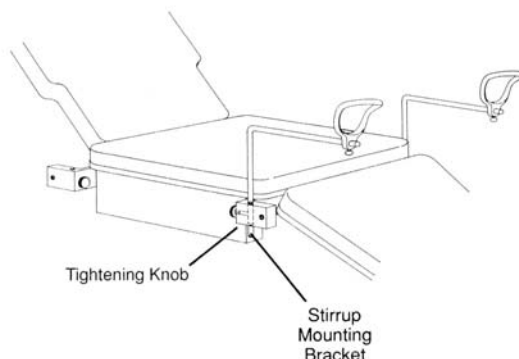

NOTE: The portion of the rod with the flat edge should be on the bottom of the horizontal rod.

3. Loosen the knobs on the bottom of the stirrups and slide them onto the horizontal rods. The knobs can be tightened at the desired position.
4. To position the stirrups:
 - a. Put the patient on the table.
 - b. Put the patient's heels into the heel cup of each stirrup.
 - c. Adjust the height and angle of the rod.
 - d. Adjust the stirrup position on the rod.