



Radiation Products Design, Inc.

Item 253-202 & 253-204 Universal Straps Directions for Use and Care

Cautions

1. Federal law restricts this device to sale by or on the order of a physician or licensed health care practitioner.
2. Select and use a restraint of adequate length and width to safely complete any procedure.
3. Examine any restraint before and during use to assure that it is in good condition and the securing mechanism is operating properly. Replace any restrains as necessary.
4. When using restrains incorporating a VELCRO® - type (hook and loop) securing material, be sure to push hook fastener side firmly into opposing component or changes in patient's condition.
5. Always monitor patients to whom restrains on any type have been applied to help avoid death or serious injury resulting from unexpected motions or changes in patient's condition.

Care and Cleaning

If necessary, material may be cleaned by hand using a cloth dampened with a mild detergent dissolved in water. DO NOT SOAK MATERIAL. For stubborn spots or stains, use a soft brush and the solution. Allow material to air dry fully before reuse.

**5218 Barthel Industrial Drive NE – Albertville, MN 55301
Phone: 763-497-2071 or 800-497-2071 Fax: 763-497-2295**