



Expect Service

Radiation Products Design Inc

INSTRUCTIONS

RPD INFORMATION

Address	5218 Barthel Industrial Drive Albertville, MN 55301
Website	www.rpdinc.com
Email	sales@rpdinc.com
Phone	763-497-2071 or 800-497-2071
Fax	763-497-2295

RPD PRODUCT INFORMATION

Item Number	Description
895-400	Varian Type III. Mini Electron Cone Adapter for 15cm

IMPORTANT

This Product Is Not Sterile To Be Used By Authorized Personnel Only

Radiation Products Design, Inc. Assumes no liability for consequential damages of any kind for this system when used or for any direct or indirect consequences of its use or misuse by their customer.

ACRYLIC CONES WITH CHROMED BRASS RING

Cleaning Use detergent similar to Klenzyme®, Manu-Klenz® or NpH-Klenz® and water or vinegar to remove hard water deposits.

WARNING DO NOT clean with alcohol as this may cause acrylic to fracture

DO NOT clean with plastic polish

DO NOT steam autoclave

Sterilization Sterrad 100NX, Steris, or Gas.
Temperature **must not** exceed 140°

CHROME STEEL ADAPTER PLATE

Cleaning Use detergent similar to Klenzyme®, Manu-Klenz® or NpH-Klenz® and water.

Sterilization Steam Autoclave, Sterrad 100NX, Steris, or Gas.

GAS STERILIZATION PROCEDURES

Warm up phase is one hour at 130°F. Conditioning phase at 130°F for 30 minutes. Inject gas, type 12% Ethylene Oxide and 88% Freon, and hold for 2 hours or type 100% Ethylene Oxide and hold for 225 minutes at 85°F or 62 minutes at 130°F. Mechanically aerate for 40 hours at 120°F or 20 hours at 130°F or 10 hours at 140°F or room aerate for 7 days.

**This Page Intentional
Left Blank**