



Expect Service

Radiation Products Design Inc

INSTRUCTIONS

RPD INFORMATION

Address	5218 Barthel Industrial Drive Albertville, MN 55301
Website	www.rpdinc.com
Email	sales@rpdinc.com
Phone	763-497-2071 or 800-497-2071
Fax	763-497-2295

RPD PRODUCT INFORMATION

Item Number	Description
192-219	Table Pad, Polyfoam, 19"x 72"x 1"
192-224	Table Pad, Polyfoam, 24"x 72"x 1"

CLEANING OF SOILS AND STAINS

Wash with a generous application of neutral soap suds and lukewarm water.

or

Use a diluted bleach solution of 10-parts water to 1-part bleach. Wipe dry with a clean cloth. Do not immerse. Bleach MUST be diluted to the 10:1 ratio.

DISINFECTING

Use Lysol I.C. Disinfectant Spray by National Laboratories, Montvale, NJ, per manufacturer's instructions.

or

Use a diluted bleach solution of 10-parts water to 1-part bleach. Wipe dry with a clean cloth. Do not immerse. Bleach MUST be diluted to the 10:1 ratio.

DO NOT APPLY TAPE TO ANY PAD

STERILIZATION

The following Steris products are also approved for use in cleaning and disinfecting the pads:

- Steris Coverage Plus Wipes 1608-WC, -GS, -G4
- Steris Coverage Spray HB Plus 1624-77
- Steris Coverage Spray TB Plus 1629-B4, -08

End of Document