



# Radiation Products Design, Inc.

## Safety Harness Instructions

Item# 495-606

1. Secure patient in the harness (a). Patient steps onto adjustable rotating platform. Seat is adjusted for height. Hands are placed on rotational hand rail assembly
2. Belt with clasp is snapped into receiver at top of rotational hand rail assembly. Pull belt snug. This easy release seat belt latch allows for fast removal of patient and will hold up to 300 pounds.

3. **Mounting Instructions:**

Tongue assembly is mounted to the 1/2" diameter steel support rod on the rotational hand rail assembly (b). Remove the 4 screws from the collars on tongue. Place on the 1/2" diameter steel support rod. Replace collars using the screws, coat with blue loctite. The loctite prevents the screws from loosening.



5218 Barthel Industrial Drive N.E. - Albertville, MN 55301

Phone: 763-497-2071 / 800-497-2071 Fax: 763-497-2295

[www.rpdinc.com](http://www.rpdinc.com)

e-mail: [sales@rpdinc.com](mailto:sales@rpdinc.com)