



Assembly Instructions for Item 244-360 Vacuum Bag Storage Rack



Carton Contents:

- | | |
|--------------------------------|---------------------------------|
| (4) Casters | (2) Horizontal Supports (Short) |
| (1) Base Assembly | (30) Nylon Support Straps |
| (4) Vertical Uprights | (1) Vertical Spacer Bar |
| (8) Horizontal Supports (Long) | (22) 1/4-20 Nylon Thumb Screws |
| (1) Top Assembly | |

Check contents of box carefully for missing pieces.

Step 1

Place (1) Base Assembly on a flat level floor area (Figure 1). Mount the (4) Casters to the bottom of the Base Assembly.



Figure 1

Step 2

Insert the (4) Vertical Uprights into the four corner holes of the Base Assembly, making sure to press each Vertical upright to the bottom of each hole. Use a 50/50 water and liquid soap mixture for lubrication if necessary (Figure 2).



Figure 2

Step 3

Install Nylon Thumb Screws on each end of the Long and Short Horizontal supports. Place (2) Long Horizontal Supports over Vertical Uprights on each side with Nylon Thumb Screw facing out and (1) Short Horizontal Support on each end. (Figure 3).

Do not tighten thumb screws.

Use the Vertical Spacer Bar to ensure even spacing between shelves then tighten the Thumb Screws.



Figure 3

Step 4

Place (2) more Long Horizontal Supports over Vertical Uprights on each side with Nylon Thumb Screw facing out.

Do not tighten thumb screws.

Place Top Assembly on Vertical Supports, using soap/water mixture if needed (Figure 4).

Use Vertical Spacer Bar to ensure even spacing between shelves then tighten thumb screws.



Figure 4

Step 5

Attach (30) Nylon Support Straps (5 on each shelf) equally spaced as needed (Figure 5) with snap heads facing the floor (Figure 5a).

Overall Dimensions:

88"L x 22"W x 73"T

Weight: 65 lbs



Figure 5

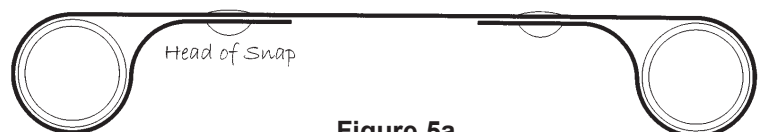


Figure 5a