



Expect Service

Radiation Products Design Inc

INSTRUCTIONS

RPD INFORMATION

Address	5218 Barthel Industrial Drive Albertville, MN 55301
Website	www.rpdinc.com
Email	sales@rpdinc.com
Phone	763-497-2071 or 800-497-2071
Fax	763-497-2295

RPD PRODUCT INFORMATION

Item Number	Description
244-500	SecureFoam, Foaming Agent - Small
244-502	SecureFoam, Foaming Agent - Medium
244-504	SecureFoam, Foaming Agent - Large

INSTRUCTIONS

1. Always wear chemical resistant gloves (latex, butyl rubber, nitrile rubber, polyvinyl chloride). Protective clothing and safety glasses or goggles required.
2. Pour contents of bottle A into bottle B.
3. Recap bottle B securely.
4. **To insure optimum foam quality, point bottle away from you and shake vigorously for 20 seconds.**
5. Carefully remove cap and immediately pour contents into the plastic bag that is provided.
WARNING: DO NOT RECAP BOTTLE B AFTER COMPLETING THE PRECEDING STEPS.
Contents remaining in bottle B will expand.
6. Close and fold the opened end of the bag. Manipulate foam to desired location with your hands on top of the bag. Once the foam begins to rise up in the bag, you are ready to position the patient onto the bag. Use the patient comfort liner for extra protection against heat.
7. Allow 8 to 10 minutes for foam to cure after initial set. Patient should remain still this entire time. Once the foam has fully hardened, remove patient from form. The form is now customized to your patient's anatomy and now ready to use. This customized form can be used several times throughout the patient's entire treatments.