



Expect Service

Radiation Products Design Inc

---

## INSTRUCTIONS

---

### RPD INFORMATION

<b>Address</b>	5218 Barthel Industrial Drive Albertville, MN 55301
<b>Website</b>	<a href="http://www.rpdinc.com">www.rpdinc.com</a>
<b>Email</b>	<a href="mailto:sales@rpdinc.com">sales@rpdinc.com</a>
<b>Phone</b>	763-497-2071 or 800-497-2071
<b>Fax</b>	763-497-2295

### RPD PRODUCT INFORMATION

<b>Item Number</b>	<b>Description</b>
467-102	Securline Pen, Each

## INSTRUCTIONS

1. Remove oil from patient's skin with alcohol or soap and water.
2. Mark incision(s) location(s) as necessary.
3. Prepare skin with Povidone - iodine (Betadine) solution to disinfect. Ink markings will fade slightly but remain legible.
4. Locate incision directly next to marked line.
5. Remove markings with Isopropyl (rubbing) alcohol if desired. Markings will disappear naturally with bathing in a few days.

**Note:** Skin marking can be done after iodine preparation if desired.