



Radiation Products Design, Inc.

**DELCLOS UTERINE/VAGINAL APPLICATOR
970-670-01, 970-670-02, 970-670-03 and 970-670-04**

**THIS PRODUCT IS NOT STERILE
TO BE USED BY AUTHORIZED PERSONNEL ONLY**

RADIATION PRODUCTS DESIGN, INC. assumes no liability for consequential damages of any kind for this material when used or for any direct or indirect consequences of its use or misuse by their customer.

ACRYLIC STERILIZATION TECHNIQUES

- Cleaning:** Use detergent similar to Klenzyme®, Manu-Klenz® or NpH-Klenz® and water or vinegar and water.
Clean with ultrasound cleaner.
- Sterilization:** Sterrad
Steris
Gas
Temperature **must not** exceed 140°
- Warning:** DO NOT clean with alcohol as this may cause acrylic to fracture.
DO NOT steam autoclave

STAINLESS STEEL STERILIZATION TECHNIQUES

- Cleaning:** Use detergent similar to Klenzyme®, Manu-Klenz® or NpH-Klenz® and water.
Warning - DO NOT clean with hydrogen peroxide or glutaradehde (Cydex).
Clean with ultrasound cleaner.
- Sterilization:** Sterrad
Steris
Gas

STERILIZATION TRAY

Steam sterilization at normal cycle
DO NOT Flash - flash steam sterilization may cause nylon coating to flake off.
Not certified for Sterrad.

**5218 Barthel Industrial Drive NE – Albertville, MN 55301
Phone: 763-497-2071 or 800-497-2071 Fax: 763-497-2295**