



Expect Service

Radiation Products Design Inc

5218 Barthel Industrial Drive, Albertville, MN 55301

Phone: (763) 497-2071 Fax: (763) 497-2295

www.rpdinc.com

WHY SHOULD YOU TEST YOUR CIDEX® SOLUTION ?

To ensure each CIDEX® Solution efficiently destroys microorganisms, the minimum effective concentration (MEC) required for high-level disinfection should be tested with the appropriate CIDEX® Test Strip prior to each usage.

DISCARD ANY UNUSED TEST STRIPS 90 DAYS AFTER BOTTLE IS ORIGINALLY OPENED.

CIDEX® OPA Test Strips

CIDEX® OPA SOLUTION CAN BE USED OR REUSED FOR A MAXIMUM OF 14 DAYS.*

How to test: Completely submerge indicating pad of the strip into solution. Hold for 1 second and remove. Do not shake strip.

Remove excess solution by standing the strip upright on a paper towel.

Read results at 90 seconds. Do not read past 90 seconds. Pad will be completely PURPLE to indicate effective solution. IF ANY BLUE remains on the pad apart from the top line, solution is ineffective and must be discarded.

*CIDEX® OPA Solution may be used or reused for up to a maximum of 14 days provided the required condition of *ortho*-phthalaldehyde concentration and temperature exist based upon monitoring described in the Instructions for Use. Do not rely solely on days in use.

PASS



FAIL



FAIL



Reorder #	Product description	Case contents
20392	CIDEX® OPA Test Strips	2 bottles – 60 strips per bottle