



**Expect Service**

## INSTRUCTIONS FOR USE

### Baumrucker Incontinence Clamp

**Item # 467-490**

#### Washing Instructions:

- Wash with a mild soap in warm water
- Rinse thoroughly in cool clean water
- Gently squeeze foam to discharge excess water
- Let unit air dry in cool environment away from excess heat or direct sunlight
- Do not use bleach, detergent, hot water, washer/dryer or blow-dryer




#### Instructions for Use


1. Apply talcum powder to skin of penis, and to clamp pads before applying.
2. Place penis between the two open sides of the clamp. Be sure the two rubber pads are beneath the penis. See figure 1 and 2. Take care not to include pubic hair in the clamp.
3. Then pull Velcro strap over and press to top side of clamp. Make sure that resulting pressure on penis is just enough to stop dribbling - DO NOT OVER COMPRESS. See figure 1 and 3.
4. Release clamp at least every 2 - 3 hours to drain bladder, and reapply following above steps 2 thru 4.
5. When resting or sleeping, clamp should be applied with less pressure than when upright and active.
6. Keep clamp dry and clean. Clamps should be alternated with a spare at least every 24 hours. Repeat step 1 above before reusing clamp.
7. This product should be used only on the advice, and under the supervision of a physician. This clamp is not intended for use in conjunction with other incontinence devices, or with indwelling catheters. This device should not be used in proximity with open sores.



**Fig. 1**

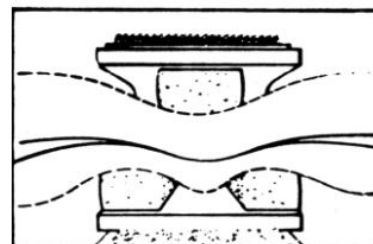

**Fig. 2**

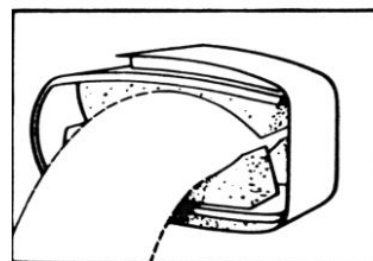

**Fig. 3**



INFO.Q 467-490-A 04/16

**5218 Barthel Industrial Drive N.E.  
Albertville, MN 55301**

**www.rpdinc.com**

**Phone: 763-497-2071 or 1-800-497-2071**

**Fax: 763-497-2295**

**Email: sales@rpdinc.com**