



Expect Service

ASSEMBLY INSTRUCTIONS

Tungsten Adjustable Electron Eyeshield for Iris

Item 936-925

Short Support Rod
936-925-1

Clamp
10-098

Long Support Rod
936-925-3

1/8" Plastic Rod, 5" Long
W/ Tungsten
936-925-4

Tungsten
69-919
↓ .400" 12 mm Stock Dia
x 10 mm Long

Drill
(Specially dulled drill for
cerrobend)
90-01167071

Tap #1/4" x 20 Thread
90-TL74540204

Tap T-Handle
90-25605A63

Cerrobend insert

1. Drill hole in cerrobend
2. Tap hole in cerrobend insert
3. Place clamp on support rod
4. Screw support rod into cerrobend
5. Insert tungsten shield rod into clamp
6. Position tungsten shield rod

Optional - Place white dot using white-out on the bottom of the tungsten shield and have patient focus on the white dot during treatment

INFO.Q 936-925-B 03-16

5218 Barthel Industrial Drive N.E.
Albertville, MN 55301

www.rpdinc.com

Phone: 763-497-2071 or 1-800-497-2071

Fax: 763-497-2295

Email: sales@rpdinc.com