



Radiation Products Design, Inc.

MAGNIFICATION RULERS

Magnification Rulers, consist of 15 scales and is graduated in magnification from 1.0 to 1.75 cm. Ideal for use with simulation or port films.

1. Identify a known measurement using either the collimator setting or a magnification marker after the film is taken.
2. Find the corresponding magnification scale for that dimension. I.E. if one side of the field edge is 10 cm as set on the collimator, match the scale, which will read 10 cm when super-imposed on that field edge. For example, begin by placing the 1.35 scale over the field edge that is 10 cm. If this scale reads less than or greater than 10 cm select the next scale which will read 10 cm.
3. The corresponding scale of the film magnification will allow measurements any place on the film.

5218 Barthel Industrial Drive NE – Albertville, MN 55301
Phone: 763-497-2071 or 800-497-2071 Fax: 763-497-2295