





Radiation Products Design, Inc.

Lifting of L-Block Shield Item 990-288 and 990-488

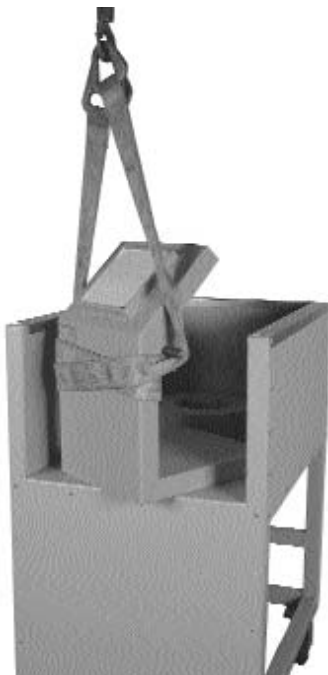

When Lifting the 450 lb, "L" Block use a sling and a 7/8" dia. x 21" long steel bar. Wrap sling around "L" Block and then around steel rod so when unit is lifted it only tilts a small amount. Bring "L" Block into Work Station at 45° so steel bar can be removed. Remove sling and steel bar, then slide unit into position.

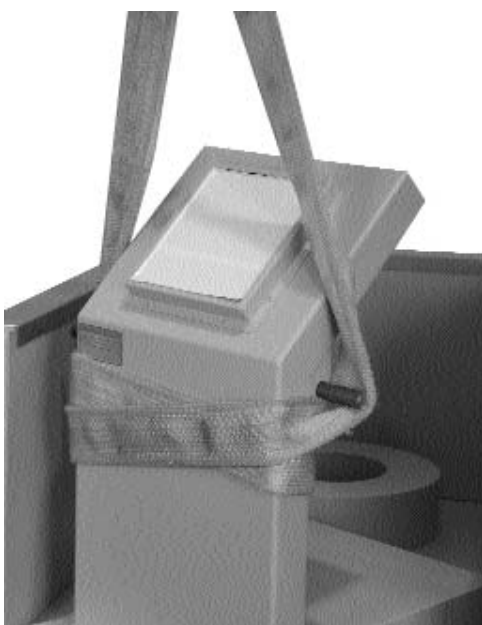

View of sling wrapping and steel bar for lifting. The steel bar is slid through the two holes in the "L"-Block.

5218 Barthel Industrial Drive N.E. Albertville, MN 55301
Phone: 763-497-2071 / 800-497-2071 Fax: 763-497-2295
www.rpdinc.com e-mail: sales@rpdinc.com