



Radiation Products Design, Inc.

Item 999-041 Flourescent Lamp Installation Instructions

To attach the lamp to the top of the safe,

Remove table clamping.

Remove the eye-bolt from the top of the radiation safe.

Install the lamp holder stud (Item 999-044) into the top of the radiation safe.

Place the lamp unit into the lamp holder stud protruding from the safe.

Adjust the stud set screw in so the unit will rotate but cannot be removed.



INFO 999-041-A 06-01

5218 Barthel Industrial Drive N.E. Albertville, MN 55301
Phone: 763-497-2071 / 800-497-2071 Fax: 763-497-2295
www.rpdinc.com e-mail: sales@rpdinc.com