

CLEANING

Lead Carrying Case Item 929-001

1. Remove excess spill with a damp cloth. Clean with a 1:1 mixture of Ivory liquid soap and water. Next, rinse with clean water and dry.
2. To disinfect use a 1:4 mixture of bleach and water. Rinse with clean water and dry.

This information is not a guarantee and does not relieve the user from the responsibility of the proper and safe use of cleaning agents. Radiation Products Design Inc. assumes no responsibility resulting from the use of other cleaning agents.

Ivory® soap is a trademark of "The Procter & Gamble Company".