



Expect Service

Radiation Products Design Inc

INSTRUCTIONS

RPD INFORMATION

Address 5218 Barthel Industrial Drive
Albertville, MN 55301

Website www.rpdinc.com

Email sales@rpdinc.com

Phone 763-497-2071 or 800-497-2071

Fax 763-497-2295

RPD PRODUCT INFORMATION

Item Number	Description
463-000	Anti-Spill Carfusion Dye Holder
464-040	Carfusion Dye, 4 oz (118 mL) (Quantity 6)
464-040-1	Carfusion Dye, 4 oz (118 mL) (Quantity 1)
464-080	Pointed Wooden Carfusion Dye Applicators (Quantity 100)



DISCLAIMER

THESE PRODUCTS ARE NOT STERILE AND ARE TO BE USED BY AUTHORIZED PERSONNEL ONLY.

RADIATION PRODUCTS DESIGN INC assumes no liability for consequential damages of any kind for this material when used interchangeably with products of other manufacturers/suppliers or for any direct or indirect results and consequences of its use or misuse by the customer.

PREPERATION FOR USE

Shake well before use.

Unscrew the caps of Item 464-040, the Carfusion Dye/Castellanis Paint and Item 463-000, Anti-Spill Carfusion Dye Holder.

Pour the Carfusion Dye/Castellanis Paint into the Anti-Spill Holder up to 1" to 1.5" from bottom. Screw the caps back on both the Carfusion Dye/Castellanis Paint and Anti-Spill Carfusion Dye Holder.

INSTRUCTIONS FOR USE

Carfusion Dye/Castellanis Paint is a long lasting skin marking dye. It dissolves skin oils and penetrates into the pores and will resist mild washings. It is recommended to do a small skin test for 24 hours before using over a large area.

Always wear gloves when using the Carfusion Dye/Castellanis Paint.

PROCEDURE – To Mark Patient

Insert one Pointed Wood Stick Applicator, Item 464-080, into the hole on the top of the Anti-Spill Carfusion Dye Holder, Item 463-000.

Remove the Applicator Stick from the Anti-Spill Carfusion Dye Holder and mark patient with applicator (like using a pencil/pen).

When the Applicator Stick is no longer leaving paint on patient – put the applicator stick in a trash receptacle.

If more marking on patient is required repeat the above steps.

NEVER RE-INSERT Applicator Stick into the Anti-Spill Carfusion Dye Holder after use on a patient.

WARNING

On rare occasion, a patient's skin could react to this chemical, forming small water blister where the dye was applied.

DANGER POISON:

Contains ethanol denatured with methanol.

May be FATAL if ingested

Refer to MSDS for further information



REMOVAL

Easy removal of Carfusion Dye/Castellanis Paint is accomplished by using Erado-Sol Stain Remover (Item 466-00 or 466-100).