



Expect Service

Radiation Products Design Inc

---

## PHYSICAL SUMMARY

---

### RPD INFORMATION

**Address** 5218 Barthel Industrial Drive  
Albertville, MN 55301

**Website** [www.rpdinc.com](http://www.rpdinc.com)

**Email** [sales@rpdinc.com](mailto:sales@rpdinc.com)

**Phone** 763-497-2071 or 800-497-2071

**Fax** 763-497-2295

### RPD PRODUCT INFORMATION

**RPD is an authorized distributor**

<b>Item Number</b>	<b>Description</b>
486-405	Super-Flex Bolus - 0.5cm x 35cm sq
486-410	Super-Flex Bolus - 1.0cm x 35cm sq
486-415	Super-Flex Bolus - 1.5cm x 35cm sq
486-420	Super-Flex Bolus - 2.0cm x 35cm sq
486-425	Super-Flex Bolus - 2.5cm x 35cm sq
486-430	Super-Flex Bolus - 3.0cm x 35cm sq
486-440	Super-Flex Bolus - 4.0cm x 35cm sq
486-505	Super-Flex Bolus with Film 0.5cm x 30cm sq
486-510	Super-Flex Bolus with Film 1.0cm x 30cm sq
486-515	Super-Flex Bolus with Film 1.5cm x 30 cm sq
486-520	Super-Flex Bolus with Film 2.0cm x 30 cm sq
486-525	Super-Flex Bolus with Film 2.5cm x 30cm sq
486-530	Super-Flex Bolus with Film 3.0cm x 30cm Sq.



**Action Products, Inc.**

22 N. Mulberry Street  
Hagerstown, MD 21740 USA

### Physical Properties Testing

**making a material difference™**

Action Products, Inc. is committed to developing the best products possible for its customers. As a result of this ongoing commitment, we are continuously evaluating and improving products, engineering better manufacturing techniques, and finding new applications for the unique and beneficial properties of Akton® polymer. Laboratory analysis of AKTON polymer, as tested under ASTM guidelines, illustrates why AKTON polymer is the best choice for shear reduction and pressure distribution in the medical field.

The rigors of elongation testing give us a glimpse at AKTON polymer's structural strength as a cross-linked solid. As described in ASTM D 412-98a\*, a prescribed specimen of medical grade AKTON polymer was stretched to its limits. The results show that AKTON polymer has an ultimate elongation of over 1000%. This remarkable degree of elasticity is a significant factor in the durability and long life of our pads and positioners.

Another important physical property that gives AKTON polymer lasting durability is its Compression Set characteristic. Test method ASTM D 395-97\* shows that after being compressed to 50% of its initial thickness for 22 hours, AKTON polymer always returned to 100% of its original size. This feature enables AKTON polymer to perform to its maximum capacity for many years of continuous use.

But the major benefits of our products are its tissue friendly characteristics. As a cushioning agent, tests show that there is minimal resistance \*, .63 lbs. per square inch, when compressing AKTON polymer a distance of 25% of its thickness. This effect is called load deflection. The initial softness allows the body to recede into the surface of the pad, achieving maximum body to pad interface. Once the body is at rest, the polymer acts to cradle and support, equalizing and distributing pressure (at 50%, the deflection force is 3.45 psi.). In addition, the softness of the material supporting the body also helps to reduce damaging shear forces. In combination, these properties are excellent evidence that products made with AKTON polymer are an ultimate choice in pressure sore prevention.

\*See the attached Smithers Scientific Services, Inc. lab report.

301.797.1414 • 800.228.7763

Fax 301.733.2073 • [www.actionproducts.com](http://www.actionproducts.com) • [email.service@actionproducts.com](mailto:email.service@actionproducts.com)



# Smithers Scientific Services, Inc.

425 WEST MARKET STREET • AKRON, OHIO U.S.A. 44303  
TELEPHONE: 330/762-7441 FAX: 330/762-7447

Page 1 of 3

February 15, 2000

CUSTOMER: ACTION PRODUCTS, INC.  
22 North Mulberry Street  
Hagerstown, MD 21740-

ATTENTION: Cliff Metger  
P.O. # 00RD14  
Smithers Job # 0002-07386

SUBJECT: TEST REPORT:  
The above-mentioned firm submitted one (1) sample for testing identified as "Akton®"

TESTING: PHYSICAL TESTING:  
ASTM D 412-98a Rubber Properties in Tension  
ASTM D 575-91(96) Compression Deflection, Method A  
ASTM D 395-97 Compression Set  
ASTM D 624-98 Die C Tear  
ASTM D 1054-91 Resilience Using a Rebound Pendulum

RESULTS: PHYSICAL TESTING:  
ASTM D 412-98a Rubber Properties in Tension

Sample Id: Akton®	100% Modulus (PSI)	200% Modulus (PSI)	300% Modulus (PSI)	Peak Tensile (PSI)	Ultimate Elongation (%)	Shore A Hardness (pts)
1.	2.6	3.2	4.8	16.9	959.1	N/A
2.	3.1	4.0	5.3	22.7	1060.3	
3.	2.6	3.9	5.6	18.4	1030.0	
Median	2.6	3.9	5.3	18.4	1030.0	

RESULTS: PHYSICAL TESTING:  
 ASTM D 575-91(96) Compression Deflection, Method A, 25% Deflection

Sample Id: Akton®	Lbs. per Square inch
1.	0.66
2.	0.63
3.	0.63
Median	0.63

ASTM D 575-91(96) Compression Deflection, Method A, 50% Deflection

Sample Id: Akton®	Lbs. per Square inch
1.	3.45
2.	3.10
3.	3.47
Median	3.45

RESULTS: PHYSICAL TESTING:  
 ASTM D 395-97 Compression Set, 22 hrs @ 25% Compression @ Room Temperature

Sample Id: Akton®	% Set
1.	0
2.	0
3.	0
Median	0

ASTM D 395-97 Compression Set, 22 hrs @ 50% Compression @ Room Temperature

Sample Id: Akton®	% Set
1.	0
2.	0
3.	0
Median	0

RESULTS: PHYSICAL TESTING:  
 ASTM D 624-98 Die C Tear

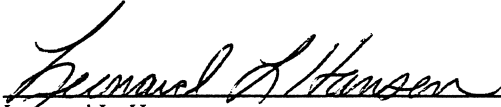
Sample Id: Akton®	Tear Strength Lb/In
1.	3.4
2.	3.5
3.	2.7
Median	3.4


RESULTS: PHYSICAL TESTING:  
 ASTM D 1054-91 Resilience Using a Rebound Pendulum  
 Specimen Size: 1.125" Diameter x 0.5" Thickness  
 Specimen Conditioning: 16 hrs @ 23°C ±2°C; 50 ± 5% Relative Humidity  
 Apparatus: Swick Model 5109 Rebound Tester

Sample Id Akton®	Rebound Resistance, R, %			Median	Average
1.	7.2	6.6	6.2	6.6	6.7
2.	9.4	9.0	8.2	9.0	8.9

*NOTE: Indentor cut through test piece on each sample; possible contact with anvil.*

*NOTE: Testing to this specification was outsourced and not performed by Smithers Scientific Services.*

  
 Leonard L. Hanson  
 Supervisor, Physical Testing/Akron Laboratories  
 Smithers Laboratories Division of  
 Smithers Scientific Services, Inc.  
 Qualified Laboratories No. 17370  
 QLL 27 31 March 1984  
 Defense Logistics Agency  
 LLH/tlw  
 U:\2000\February\07386.doc

  
 Albert L. Harrison  
 Operations Manager/Akron Laboratories  
 Smithers Laboratories Division of  
 Smithers Scientific Services, Inc.  
 Qualified Laboratories No. 17370  
 QLL 27 31 March 1984  
 Defense Logistics Agency