

Radiation Products Design Inc

INSTRUCTIONS

DDD	NEOP	MATION

Address	5218 Barthel Industrial Drive	
	Albertville, MN 55301	

- Website www.rpdinc.com
- Email sales@rpdinc.com
- Phone 763-497-2071 or 800-497-2071
- **Fax** 763-497-2295

RPD PRODUCT INFORMATION

Item Number	Description
468-016	Black Tattoo Ink, 1/2 oz., w/2 SS Balls, NON-STERILE

DISCLAIMER

THESE PRODUCTS ARE NOT STERILE AND ARE TO BE USED BY AUTHORIZED PERSONNEL ONLY.

RADIATION PRODUCTS DESIGN INC assumes no liability for consequential damages of any kind for this material when used interchangeably with products of other manufacturers/suppliers or for any direct or indirect results and consequences of its use or misuse by the customer. Federal law (USA) restricts the sale of this device for use only by (or at the order of) a physician.

INTRODUCTION

The Black Tattoo Ink contains alcohol and two stainless steel balls inside for mixing by shaking. NON-STERILE. 0.5 oz / 15 ml

INSTRUCTIONS

Always shake well before use.

Follow current department protocols regarding intra-dermal procedures.

Gently insert desired needle point (lumen) under skin surface.

Carefully squeeze dropper bottle, dripping ink onto needle tip. Twist needle and remove.

Discard needle in proper container

DO NOT tattoo near eyes or over temporal area.

End of Document