



Expect Service

Radiation Products Design Inc

INSTRUCTIONS

RPD INFORMATION

Address	5218 Barthel Industrial Drive Albertville, MN 55301
Website	www.rpdinc.com
Email	sales@rpdinc.com
Phone	763-497-2071 or 800-497-2071
Fax	763-497-2295

RPD PRODUCT INFORMATION

Item Number	Description
468-016	Black Tattoo Ink, 1/2 oz., w/2 SS Balls, NON-STERILE

DISCLAIMER

THESE PRODUCTS ARE NOT STERILE AND ARE TO BE USED BY AUTHORIZED PERSONNEL ONLY.

RADIATION PRODUCTS DESIGN INC assumes no liability for consequential damages of any kind for this material when used interchangeably with products of other manufacturers/suppliers or for any direct or indirect results and consequences of its use or misuse by the customer. Federal law (USA) restricts the sale of this device for use only by (or at the order of) a physician.

INTRODUCTION

The Black Tattoo Ink contains alcohol and two stainless steel balls inside for mixing by shaking. NON-STERILE. 0.5 oz / 15 ml

INSTRUCTIONS

Always shake well before use.

Follow current department protocols regarding intra-dermal procedures.

Gently insert desired needle point (lumen) under skin surface.

Carefully squeeze dropper bottle, dripping ink onto needle tip.

Twist needle and remove.

Discard needle in proper container

DO NOT tattoo near eyes or over temporal area.

LABEL ON BOTTLE

½ oz	BLACK	LOT # xxxx
	NOT FOR EYE AREA USE	BATCH xxxx
	IMPORTANT NOTICE	EXP xx/xx

Despite constant efforts to ensure that this product is non-toxic and meets the highest standards of quality and purity, application of this product on certain individuals may cause an ALLERGIC REACTION due to the sensitivity of the dye.

The company specifically DISCLAIMS any responsibility for the allergic reaction of any individual to whom this dye is applied whether implied by a WARRANTY OF MERCHANTABILITY OR FITNESS FOR PURPOSE or otherwise. Where there is no history of a tolerance to this dye, spot testing or consultation with a competent dermatologist prior to application is urged.

End of Document