



**Expect Service**

**Radiation Products Design Inc**

---

## INSTRUCTIONS

---

### RPD INFORMATION

<b>Address</b>	5218 Barthel Industrial Drive Albertville, MN 55301
<b>Website</b>	<a href="http://www.rpdinc.com">www.rpdinc.com</a>
<b>Email</b>	<a href="mailto:sales@rpdinc.com">sales@rpdinc.com</a>
<b>Phone</b>	763-497-2071 or 800-497-2071
<b>Fax</b>	763-497-2295

### RPD PRODUCT INFORMATION

<b>Item Number</b>	<b>Description</b>
468-105	SteriTatt 3cc Droppers, Black

## INSTRUCTIONS FOR USE

Follow current department protocols regarding intra-dermal procedures.

Single Patient use, Sterile Tattoo Ink

## DROPPING TECHNIQUE



### Step 1

Verify that SteriTatt dropper is sealed and unopened. Shake very well.



### Step 2

Twist off cap and remove heat seal.



### Step 3

Hold SteriTatt dropper between thumb and index finger. Over trash receptacle, squeeze to produce ink drops.



### Step 4

Gently insert desired needle point (lumen) under skin surface. Carefully squeeze dropper, dripping ink onto needle tip. Twist needle and remove.



### Step 5

Discard needle in Sharps container, SteriTatt dropper in normal trash.

\*For a slightly larger tattoo, use large diameter needle.

**End of Document**