



Expect Service

Radiation Products Design Inc

INSTRUCTIONS

RPD INFORMATION

Address	5218 Barthel Industrial Drive Albertville, MN 55301
Website	www.rpdinc.com
Email	sales@rpddinc.com
Phone	763-497-2071 or 800-497-2071
Fax	763-497-2295

RPD PRODUCT INFORMATION

Item Number	Description
253-202	Straps, 2" Universal for 18" to 30" Wide Tables
253-204	Straps, 4" Universal for 18" to 30" Wide Tables
253-206	Straps, 6" Universal for 18" to 30" Wide Tables

DESCRIPTION

Hook-and-loop adjustable strap is soft against the skin. It has no metal hardware, is radiolucent and artifact free.

USE

- Select and use a position strap of adequate length and width to safely complete any procedure.
- Loop under table pad or extension and around patient.
- When using positioning straps incorporating a hook-and-loop securing material, be sure to push hook fastener side firmly into opposing component and check security.

WARNINGS

Federal law restricts this device for sale by, or on the order of a physician or licensed health care practitioner.

Always monitor patients to whom positioning straps of any type have been applied to help avoid potential serious injury or death resulting from unexpected motions or changes in patient's condition.

CARE

Examine any positioning strap before and during use to assure that it is in good condition, and the securing mechanism is operating properly. Replace any positioning straps as necessary.

If necessary, material may be cleaned by hand using a cloth dampened with a mild detergent dissolved in water. **DO NOT SOAK MATERIAL.** For stubborn stains, use a soft brush and the solution. Allow material to air-dry fully before reuse.

CONSTRUCTION

Hook-and-loop strap is made of nylon.

End of Document