



Expect Service

Radiation Products Design Inc

---

## INSTRUCTIONS

---

### RPD INFORMATION

<b>Address</b>	5218 Barthel Industrial Drive Albertville, MN 55301
<b>Website</b>	<a href="http://www.rpdinc.com">www.rpdinc.com</a>
<b>Email</b>	<a href="mailto:sales@rpdinc.com">sales@rpdinc.com</a>
<b>Phone</b>	763-497-2071 or 800-497-2071
<b>Fax</b>	763-497-2295

### RPD PRODUCT INFORMATION

<b>Item Number</b>	<b>Description</b>
693-080	Leveling Plate, with Support Board, for Water Phantom Tanks

## ASSEMBLY INSTRUCTIONS

1. Lay support board on table or countertop.
2. Place the three mounting pads onto the support board.
3. Screw the knobs into leveling plate with about an inch and a half of the threaded rod protruding from under the plate.
4. Lay the leveling plate over the support board with the ends of the threaded rods landing on top of the three mounting pads.
5. Screw in the two feet having the padded ends facing downward. You should leave about a quarter inch of rod extended from the plate.
6. Level the plate by adjusting the three knobs and leave about an inch of space between the leveling plate and support board.
7. After leveling, lower the two padded feet to secure the plate in place.

