

## PET WORKSTATION TRANSPORTATION

## Lifting of L-Block Shield (Item 990-288 and 990-488)

When Lifting the 450 lb, "L" Block, use a sling and a 7/8" dia. x 21" long steel bar. Wrap the sling around the "L" Block and then around steel rod so that when the unit is lifted, it only tilts a small amount. Set the "L" Block onto the Work Station at a 45° angle so the steel bar is able to be removed. Remove the sling and steel bar, then slide unit into position.

The second image shows the sling wrapping and steel bar for lifting. The steel bar is slid through the two holes in the "L"-Block.





## **Transportation of Workstation (Item 994-020)**

The workstation is delivered with two eye bolts installed that are used to secure the workstation during transport. The eye bolts can be removed and replaced with the included 5/8" internal hex button head socket cap screw.









